REF: 1907

Spengler

MANUEL UTILISATEUR
USER'S MANUAL
MANUAL
GEBRAUCHSANWEISUNG
MANUALE UTENTE
GEBRUIKERSHANDLEIDING
KÄYTTÖOPAS
KASUTUSJUHEND
LIETOTĀJĀ ROKASGRĀMATĀ
دليل المستخدم

LAUBRY®

Stéthoscope
Fonendoscopio
مستعارة طبيب

DÉSIGNATION / DENOMINATION / التسمية	RÉFÉRENCE / REFERENZE / الرقود / المرجعي
Stéthoscope LAUBRY - noir LAUBRY stethoscope - black Estetoscopio LAUBRY - negro LAUBRY Stethoskop - schwarz Stetoskopio LAUBRY - nero LAUBRY stethoscopp - zwart LAUBRY stetoskoopji - musta LAUBRY stetoskoop - must LAUBRY stetoskops - melns	507 300
<i>مستعارة الطبيب لوبري - اسود</i>	
Stéthoscope LAUBRY - marine LAUBRY stethoscope - navy blue Estetoscopio LAUBRY - azul marino LAUBRY Stethoskop - Marineblau Stetoskopio LAUBRY - biru navy LAUBRY stethoscopp - marieblaau LAUBRY stetoskoopji - laivastonsininen LAUBRY stetoskoop - tumesininne LAUBRY stetoskops - tumli zilš	507 302
<i>مستعارة الطبيب لوبري - الأزرق الداكن</i>	
Stéthoscope LAUBRY - rouge LAUBRY stethoscope - red Estetoscopio LAUBRY - rojo LAUBRY Stethoskop - rot Stetoskopio LAUBRY - rosso LAUBRY stethoscopp - rood LAUBRY stetoskoopji - punainen LAUBRY stetoskoop - sarķans	507 308
<i>مستعارة الطبيب لوبري - احمر</i>	

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USER'S MANUAL EN

Thank you for choosing the LAUBRY stethoscope.
Choosing Spengler's instrument is trusting the experience and recognized excellence since 1907.
LAUBRY is the resulting from many years of research and development aiming at providing the best quality, design and comfort to health professionals.
This medical device is exclusively manufactured in accordance with the regulations in force, relating to medical devices and is subject to quality controls.

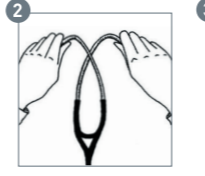
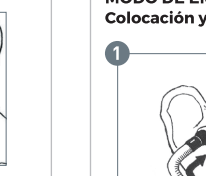
INSTRUCTIONS FOR USE
Please read carefully the user manual before using the product and keep it safe for further reference.

INTENDED MEDICAL USE
LAUBRY is a professional equipment for aid in the diagnosis. It is an acoustic instrument used for auscultation to listen to sounds within the human body in order to make a medical diagnosis: for example during heart, lungs and other organic noises auscultation as well as measuring the blood pressure.

INDICATIONS
LAUBRY stethoscope is a medical device Class I, for aid in the diagnosis, intended for health care professionals, trained for its use. It is used to listen to sounds within the body for gathering information about patient condition and to measure blood pressure for all types of patients: adults and children.

CONTRAINDICATIONS
- Do not used on injured skin.
- Do not used in case of instruments material allergies.

INSTRUCTIONS FOR USE
Fitting and adjusting the binaural tubes

The binaural tubes are designed to automatically adjust to your ears' listening angle. Ensure that the branches of the binaural tubes are pointing forwards, thus matching the direction of the auditory canal [fig. 1].
The binaural tube spring can be adjusted for improved comfort and better adjustment to your morphology.
To increase the spring tension, grasp the branches of the binaural tubes at the level of the earpieces and cross them over [fig. 2].
To reduce the spring tension, grasp the branches at the base of the binaurals and gradually move them apart to complete extension [fig. 3].

Chest-piece	Single head
Material	Zinc (Zamac)
Weight	78g
Diameter	49 mm
Diaphragm	
Material	Epoxy Resin Glass Fiber
Diameter	44 mm
Non-chill ring	
Material	PVC
Diameter	49 mm
Binaurals and other parts	
Binaurals length	62 cm
Binaurals material	Aluminium
Spring material	Steel
Tubing material	PVC
Ear tips material	TPE
Total length	79 cm
Accessories included	2 pairs of ear-tips + identification tag

WARNINGS
- Do not immerse the stethoscope in water/liquid.
- Do not use if the packaging is damaged.
- Do not sterilize.

SIDE EFFECTS
None

MATERIOVIGILANCE
All serious incidents related to the use of a medical device should be notified to the manufacturer and the responsible authority of the Member State where the user and/or the patient live.

STORAGE AND HANDLING CONDITIONS
- Keep the stethoscope away from excessive heat or cold.
- Avoid exposing stethoscope to direct sunlight.

MAINTENANCE
Earpiece and diaphragm replacement

- Earpieces**: Regularly ensure that the ends are properly clipped/screwed on to preserve good acoustic quality. To replace them, pull on the ends then replace them by pushing the new ends into place.
- Diaphragm**: The diaphragm is held in place by the chill ring. To remove the diaphragm, grip the chill ring between thumb and index finger and pull to remove it from the chestpiece [fig. 4].
Replace the diaphragm on the chestpiece, printed side up, and align the edge of the chill ring with the chestpiece groove. Using your thumbs, attach the chill ring to the chestpiece circumference [fig. 5].
If the device seems damaged or if the sounds are not audible anymore, do not use the device. Contact the after sales department.

UPKEEP AND CLEANING RECOMMENDATIONS
Regular cleaning of your stethoscope and in particular of the earpieces, ensures optimum acoustic performance.
- Clean the stethoscope with a cloth soaked in 70% isopropyl alcohol or moderately soapy water.
- Remove the earpieces before cleaning them thoroughly by soaking, then drying them. Ensure that no dirt remains inside.

DISPOSAL
Dispose of the device at the specific collection point according to local regulation.

QUARANTÉE
The LAUBRY stethoscope is guaranteed against manufacturing defects (parts and labour) under normal conditions of use, from the date of purchase (invoice date).
- Chestpiece, binaural tube assembly- 5 years
- Earpieces, diaphragm and chill rings- 1 year
The guarantee does not apply in the event of use non-compliant with instructions, improper use, modifications made by yourself or by an unauthorised third party, accidental damage and normal wear.

During the guarantee period, any stethoscope parts that present manufacturing defects shall be replaced and/or repaired free of charge, after returning the device to Spengler's after-sales service. In the event of a guarantee claim or of repairs, please return the complete stethoscope in its original packaging, accompanied by the dated invoice.

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