seca bodimeter 206

www.distrimed.com

Herzlichen Glückwunsch

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Congratulations

In the seca 206 bodimeter you have purchased an easy-to-use device for measuring the height of children and adults. It is designed for use in hospitals, doctors' surgeries and in-patient measuring and nursing facilities.

Safety

Before using the new bodimeter, please take a little time to read the following safety instructions.

- Follow the operating instructions.
- Use the mounting material supplied.
- After measuring, push the bodimeter back up so that there is no risk of injury.

Attaching the bodimeter to the wall

1. Place the bodimeter on the floor against the wall.
2. Pull the tape measure out far enough for the zero to line up exactly with the red stripe in the read-off area. Then mark the site of the drill hole on the wall in the upper part of the attachment hole.

3. Attach the end of the tape measure to the wall using the screw supplied.
4. Push the bodimeter back together.

Measuring with the bodimeter

- a. The person to be measured stands under the bodimeter.
- b. The measuring tongue is lowered onto the head.
- c. Height appears in the read-off area.

If the above-mentioned steps are followed, measurement accuracy is better than +/- 5 mm.

Cleaning

Clean the measuring rod as required using a domestic cleaning agent or commercially-available disinfectants. Follow the manufacturer's instructions.

Technical data

Measuring range: 0 - 220 cm | 87 ins
Graduations: 1 mm | 1/8 in
Dimensions (W x H x D): 35 x 115 x 150 mm approx. 140 g
Medical product as per
Directive 93/42/EEC: Class I

Disposal

If it is no longer possible to use the measuring device, your responsible waste disposal authority will be pleased to provide information about the measures necessary to dispose of it properly.

Guarantee

We guarantee defects attributable to poor materials or workmanship for 24 months from the date of purchase.

The guarantee is invalidated if the device is opened by unauthorised parties.

Important: guarantee valid only in conjunction with proof of purchase.

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